



## 1 Attach electrodes to device

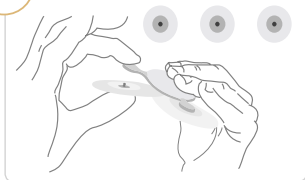

## 2 Peel off plastic from electrodes

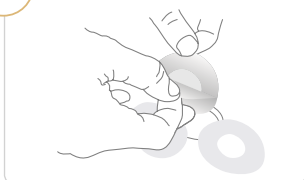

## 3 Turn on device

Press button



## 4 Attach device to patient

- Slow green flashes indicate C3 is correctly attached and recording
- The C3 will keep recording for 24h - 48h



**Please note:**  
C3 is **NOT**  
waterproof

Place C3 on midsternal line



**Please note:**  
In case of chest hair,  
shaving may be required



## A Device notification lights



### Green flashes

- Fast: ON / Event button pressed
- Slow: C3 is recording



### Blue flashes

- C3 does not have proper contact to the body



### Yellow flashes

- Slow: Low battery
- Fast: Non-critical Error (Consult Manual)



### White flashes

- Critical Error (Contact Supplier)

## B Contents of package



Cortrium C3 unit



USB-cable




Instructions for Use

## NB Discomfort!

### If sensing symptoms or discomfort, press button


- Button will flash green quickly for 10 seconds.

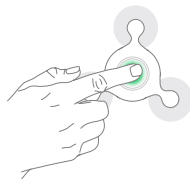

## C Charging the C3

### Please note:

Charge for at least one hour

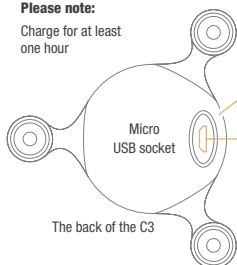

The back of the C3

USB Plug

USB Cable



Main power outlet

