Cortrium Apex 1.16

Bruksanvisning

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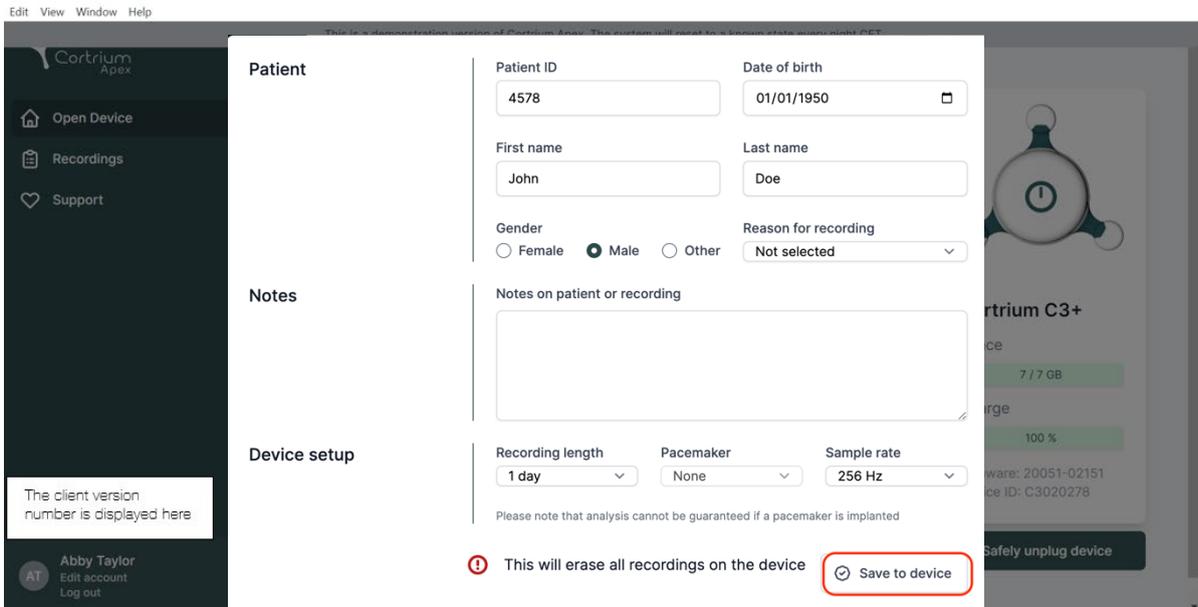
The screenshot shows the Cortrium Apex software interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Support'. The main area is a white form titled 'Patient' with the following fields:

- Patient ID:** 4578
- Date of birth:** 01/01/1950
- First name:** John
- Last name:** Doe
- Gender:** Radio buttons for Female, Male (selected), and Other.
- Reason for recording:** A dropdown menu currently set to 'Not selected'.
- Notes:** A large text area for 'Notes on patient or recording'.
- Device setup:**
 - Recording length:** 1 day
 - Pacemaker:** None
 - Sample rate:** 256 Hz

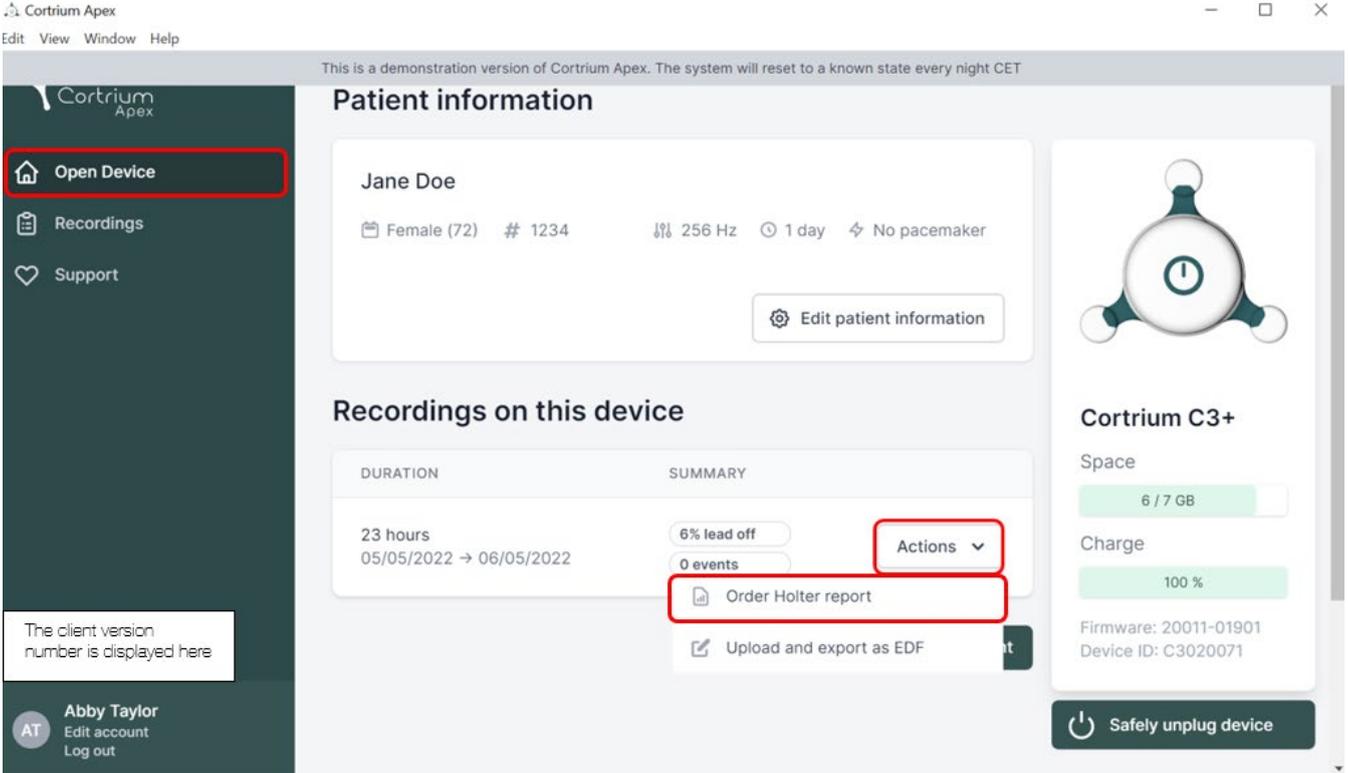
At the bottom of the form, there is a red warning icon and the text: 'This will erase all recordings on the device'. To the right of this warning is a button labeled 'Save to device' which is highlighted with a red rectangle. On the far right, a portion of another screen is visible, showing 'Cortrium C3+' and a 'Safely unplug device' button.

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The screenshot shows the Cortrium Apex web application interface. The top navigation bar includes 'Edit View Window Help' and a status message: 'This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET'. The main content area is divided into three sections:

- Patient information:** Displays 'Jane Doe', 'Female (72)', '# 1234', '256 Hz', '1 day', and 'No pacemaker'. An 'Edit patient information' button is present.
- Recordings on this device:** Shows a recording duration of '23 hours' from '05/05/2022' to '06/05/2022'. The summary includes '6% lead off' and '0 events'. An 'Actions' dropdown menu is open, with 'Order Holter report' highlighted.
- Cortrium C3+ device status:** Shows 'Space: 6 / 7 GB', 'Charge: 100%', 'Firmware: 20011-01901', and 'Device ID: C3020071'. A 'Safely unplug device' button is at the bottom.

The sidebar on the left contains 'Open Device', 'Recordings', and 'Support' buttons. A user profile for 'Abby Taylor' is visible at the bottom left.

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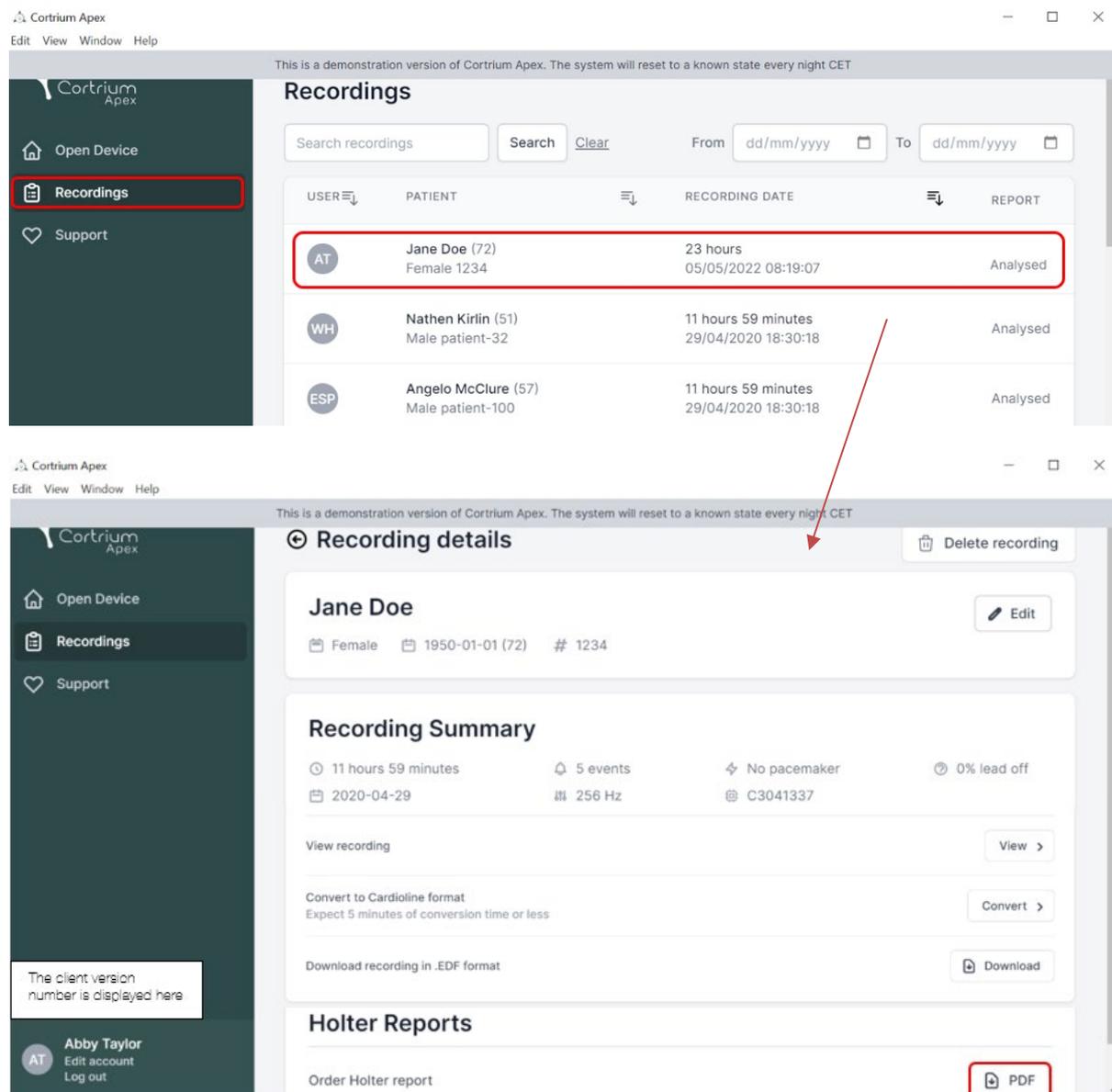
The screenshot shows two windows of the Cortrium Apex application. The top window displays the 'Recordings' list, and the bottom window displays the 'Recording details' for a specific recording.

Recordings List:

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirlin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

Recording details for Jane Doe:

Recording Summary

- 11 hours 59 minutes
- 5 events
- No pacemaker
- 0% lead off
- 2020-04-29
- 256 Hz
- C3041337

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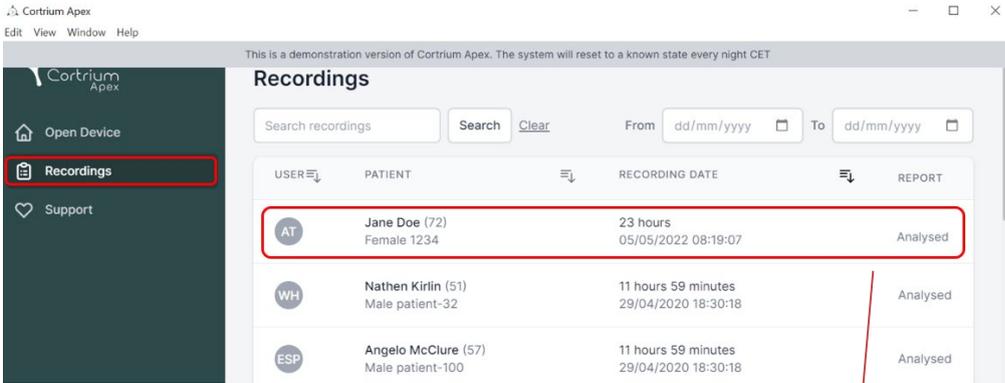
Cortrium Apex
Edit View Window Help


This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Recordings

Search recordings Search From To

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirilin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

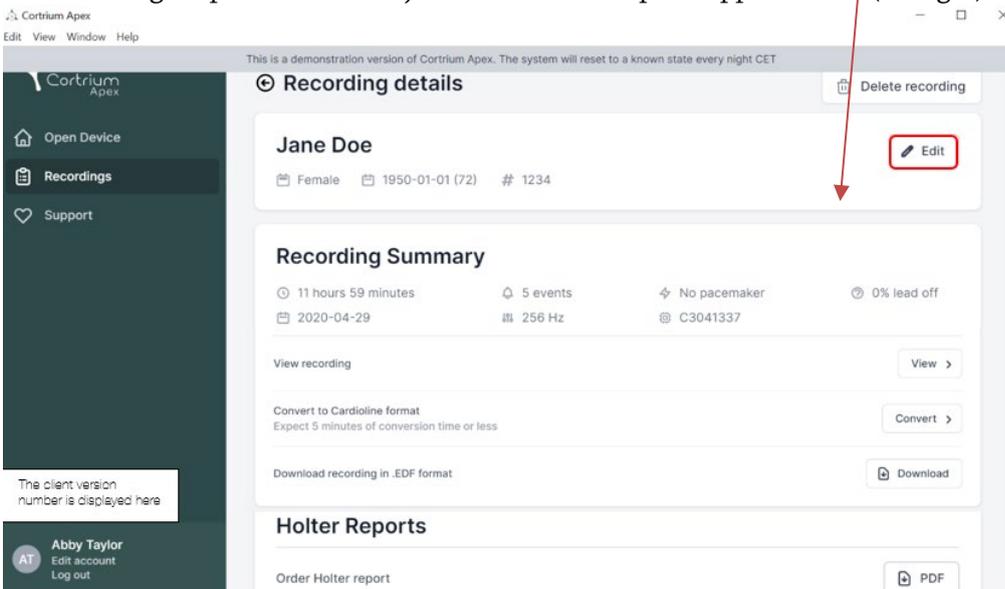Du kan redigere pasientinformasjonen ved å klikke på knappen «Edit» (Rediger).

Cortrium Apex
Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Recording details

Jane Doe
Female 1950-01-01 (72) # 1234

Recording Summary

11 hours 59 minutes 5 events No pacemaker 0% lead off
2020-04-29 256 Hz C3041337

View recording

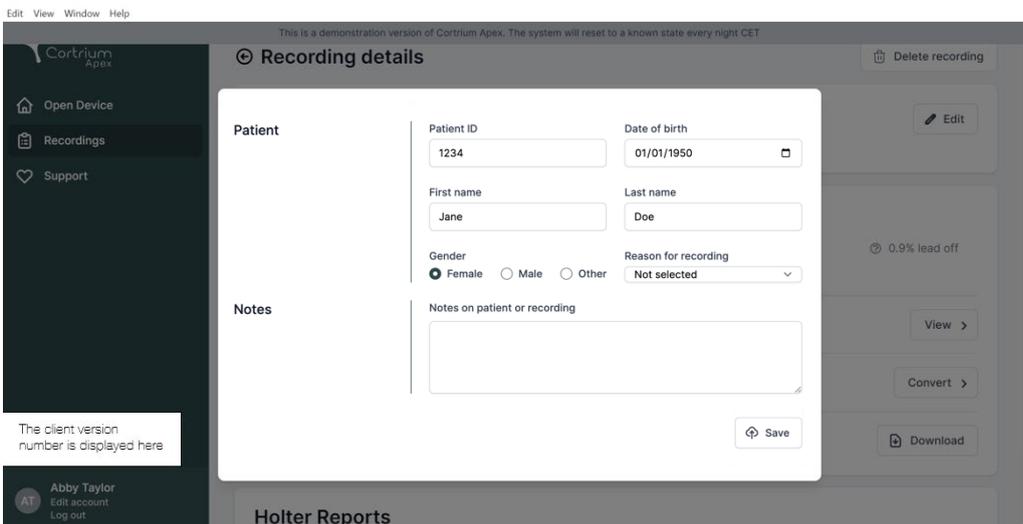
Convert to Cardioline format
Expect 5 minutes of conversion time or less

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
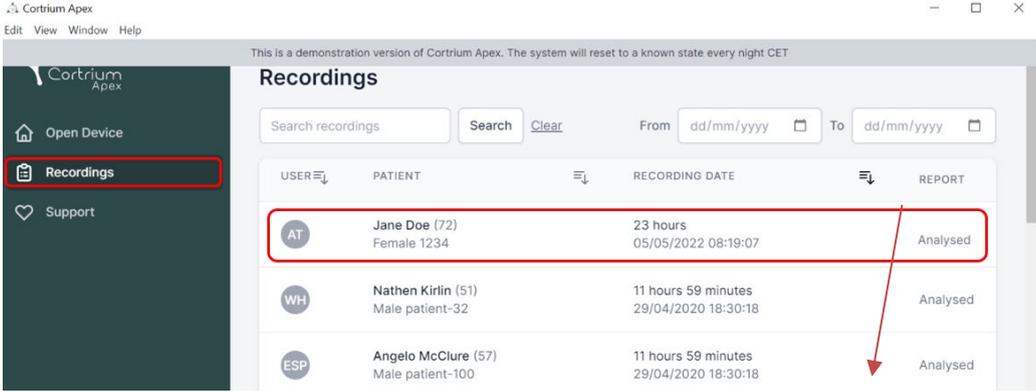
This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Recordings

Search recordings Search From To

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirilin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

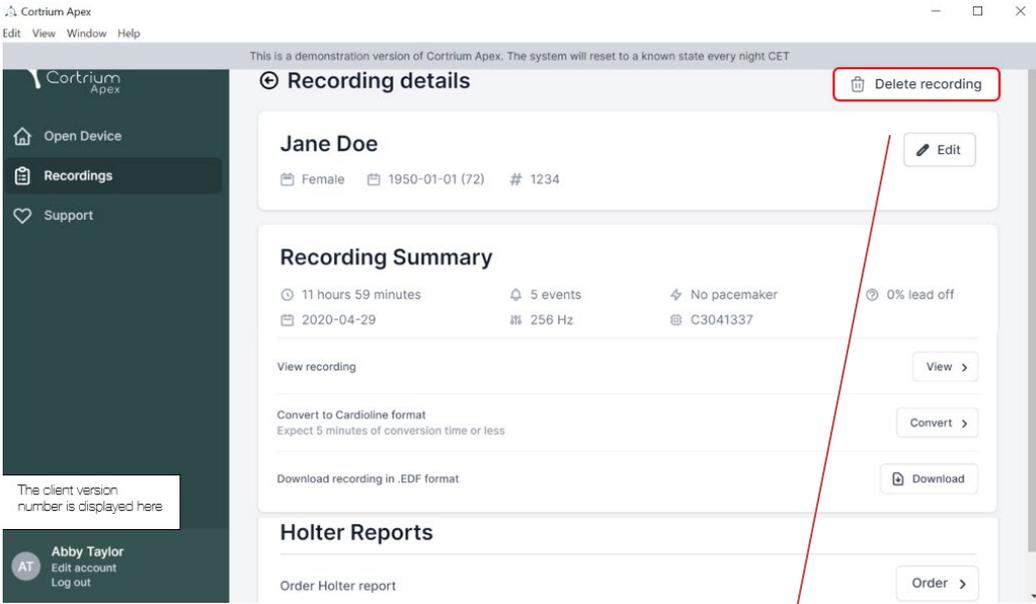This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Recording details

Jane Doe
Female 1950-01-01 (72) # 1234

Recording Summary

11 hours 59 minutes 5 events No pacemaker 0% lead off
2020-04-29 256 Hz C3041337

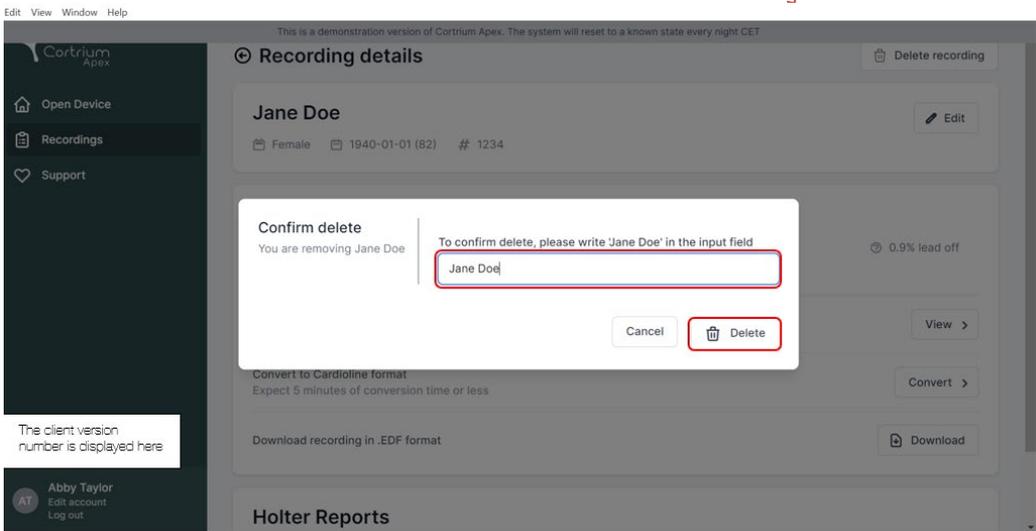
View recording

Convert to Cardioline format
Expect 5 minutes of conversion time or less

Download recording in .EDF format

Holter Reports

Order Holter report

Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Recording details

Jane Doe
Female 1940-01-01 (82) # 1234

Confirm delete
You are removing Jane Doe

To confirm delete, please write 'Jane Doe' in the input field

0.9% lead off

Convert to Cardioline format
Expect 5 minutes of conversion time or less

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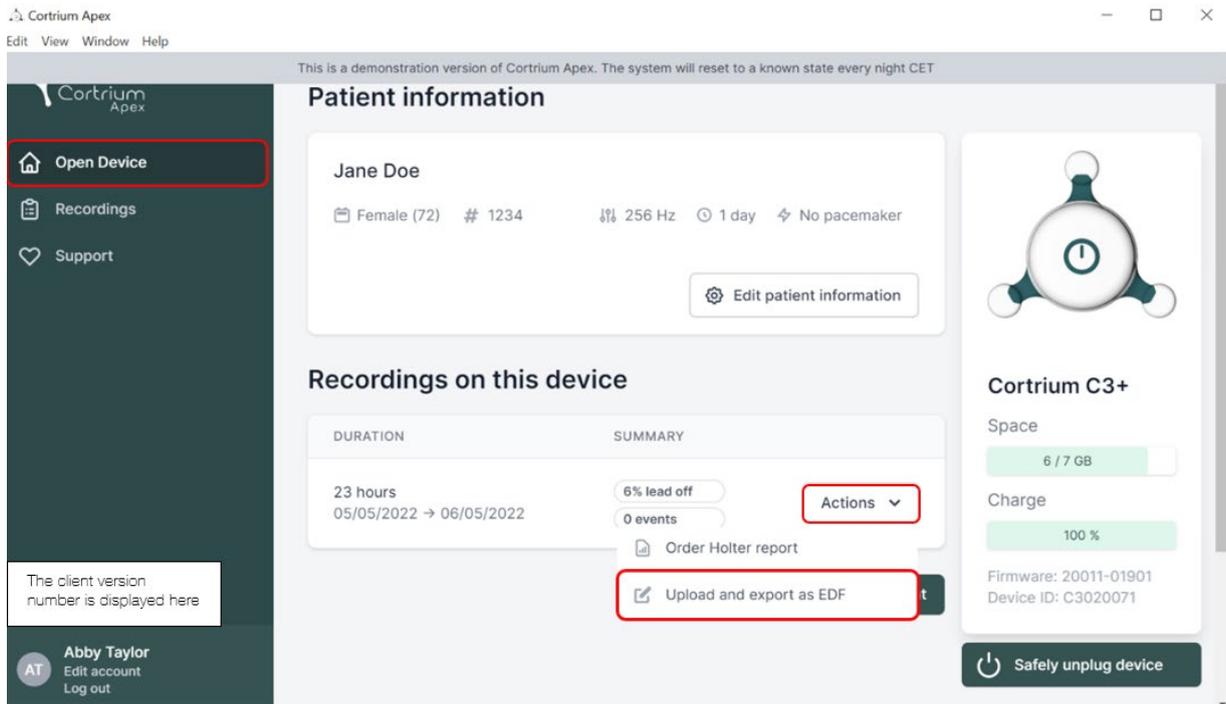
The screenshot shows the Cortrium Apex web application interface. The top navigation bar includes 'Cortrium Apex', 'Edit', 'View', 'Window', and 'Help'. A notification states: 'This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET'. The main content area is divided into three sections:

- Patient information:** Displays 'Jane Doe', 'Female (72)', '# 1234', '256 Hz', '1 day', and 'No pacemaker'. An 'Edit patient information' button is present.
- Recordings on this device:** A table with columns 'DURATION' and 'SUMMARY'. The first row shows '23 hours' for the duration (05/05/2022 → 06/05/2022) and '6% lead off' and '0 events' in the summary. An 'Actions' dropdown menu is visible, with 'Upload and export as EDF' highlighted.
- Device status (Cortrium C3+):** Shows 'Space' at 6 / 7 GB, 'Charge' at 100%, 'Firmware: 20011-01901', and 'Device ID: C3020071'. A 'Safely unplug device' button is at the bottom.

The left sidebar contains 'Open Device', 'Recordings', and 'Support' buttons. A user profile for 'Abby Taylor' is visible at the bottom left.

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Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Cortrium Apex

Open Device

Recordings

Support

The client version number is displayed here

Abby Taylor
Edit account
Log out

Patient information

Jane Doe

Female (72) # 1234 256 Hz 1 day No pacemaker

Edit patient information

Recordings on this device

DURATION	SUMMARY
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events Actions

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 12%

Cortrium C3+

Space: 6 / 7 GB

Charge: 100 %

Firmware: 20011-01901
Device ID: C3020071

Cortrium Apex Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Cortrium Apex

Open Device

Recordings

Support

The client version number is displayed here

Abby Taylor
Edit account
Log out

Patient information

Jane Doe

Female (72) # 1234 256 Hz 1 day No pacemaker

Edit patient information

Recordings on this device

DURATION	SUMMARY
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events View

Erase all and prepare for new patient

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